


Physician Instructions: During Patient Visit


Step 1: Return to MyInsights Page

The next time you log in, simply select which patient you'd like to engage, and click "Enter Data for this Patient" to begin working with their records.

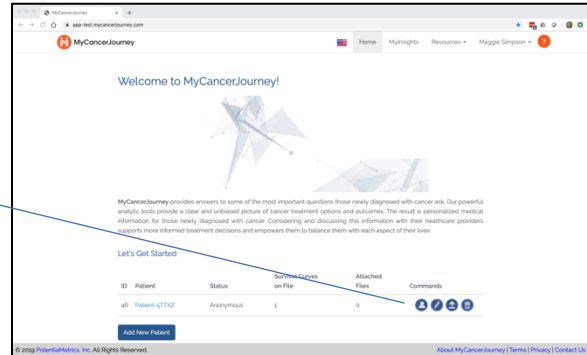


Step 2: View the Patient's Survival Curve(s)

From here, you can work with existing survival curves and/or create new ones for the same cancer site, as well as other cancer sites, in the event of additional new diagnoses.

Select the  button to the right to view an existing already-generated survival curve, or...



Select the  button at the bottom to generate new curves for new diagnoses as needed.

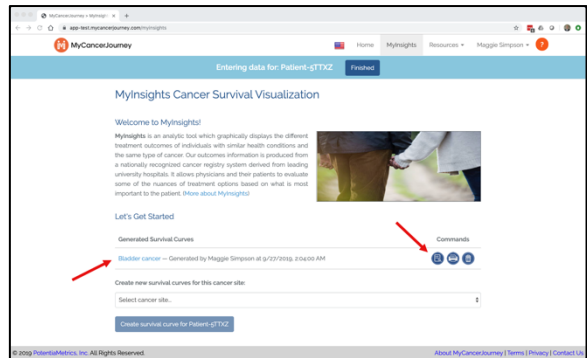

Step 4: Assemble and Print a Shared Decision-Making Report

Indicate you want to print a patient report

Having selected an existing generated survival curve, the survival curves page will reopen.


Print Shared Decision-Making Report

Select the button to the right of the curves chart.



Configure the Report

Select the sections of information that you, as a physician, want to communicate to the patient in the SDM report. Note that the system will default treatment options that are displayed in the survival curves.

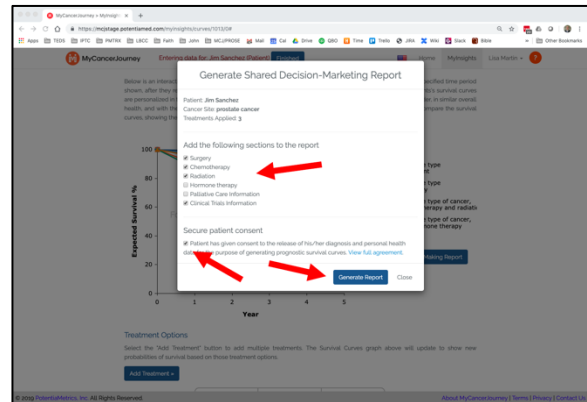


Secure Patient Consent

Make sure the patient understands the use of the SDM report. Click “View Full Agreement” at the bottom of the dialog, so that they can read the patient consent document, and then check the box to the lower left to explicitly give their approval to create the SDM report.

Generate the Report

Generate Report

Click to generate the report. It will open in a separate tab in your web browser. You may save or print the report from this point.

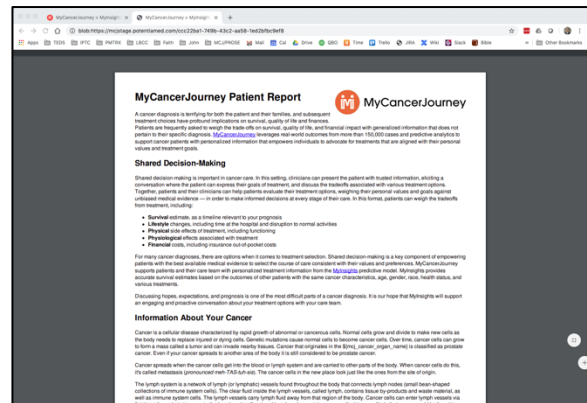

NOTE: You must enable pop-ups from MyCancerJourney.com for the report to display.